
***DONJOY***<sup>®</sup>

# ULTRASLING™ QUADRANT

ENGLISH .....	2
ESPAÑOL .....	4
DEUTSCH .....	6
ITALIANO.....	8
FRANÇAIS.....	10
NEDERLANDS .....	12
PORTUGUÊS (PORTUGAL) .....	14
PORTUGUÊS (BRASIL) .....	16
ČEŠTINA .....	18
SLOVENČINA.....	20
DANSK.....	22
SVENSKA .....	24
SUOMI .....	26
NORSK.....	28
POLSKI.....	30









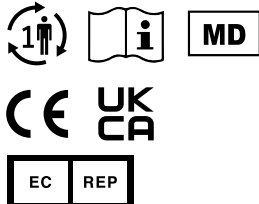
UK Responsible Person  
MDSS-UK RP LIMITED  
6 Wilmslow Road, Rusholme,  
M14 5TP Manchester,  
United Kingdom

DJO, LLC  
5919 Sea Otter Place,  
Suite 200  
Carlsbad, CA 92010  
USA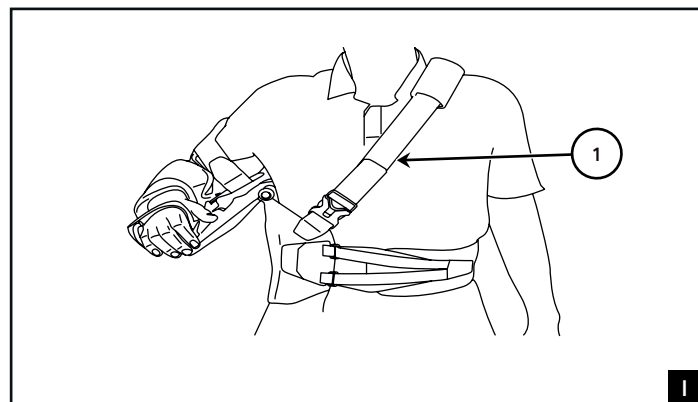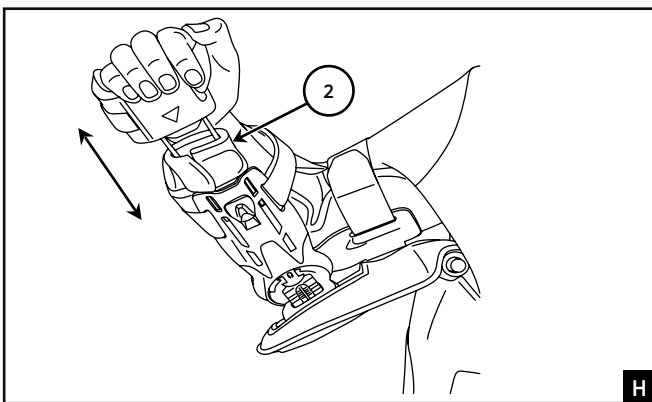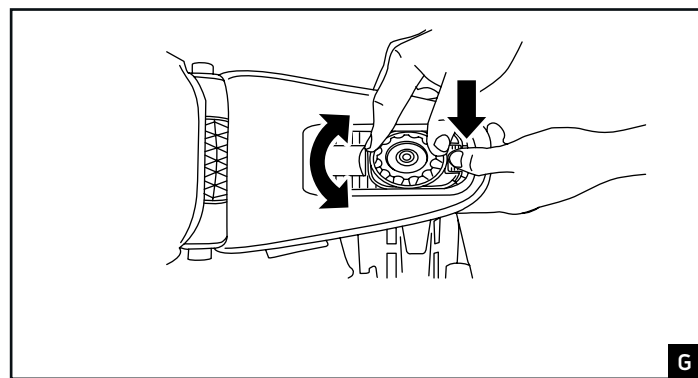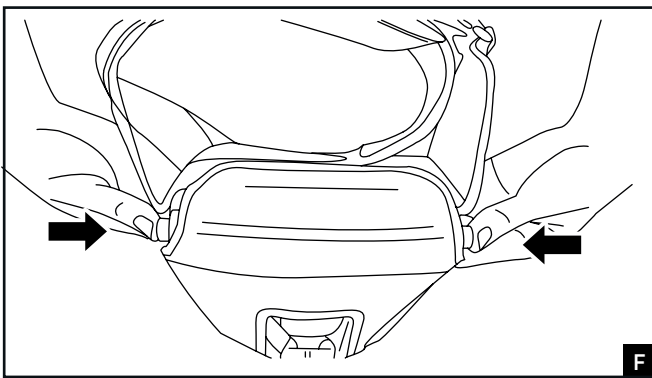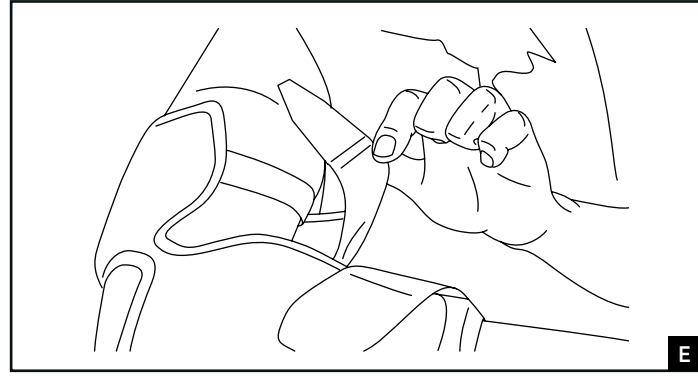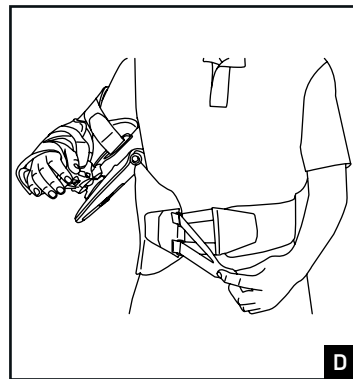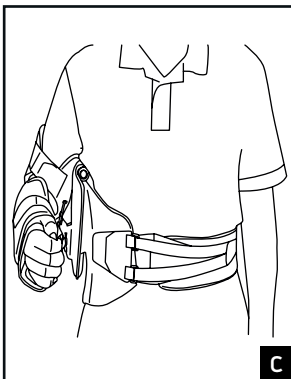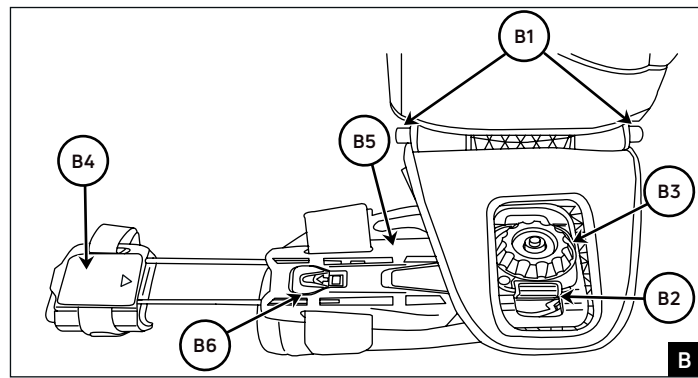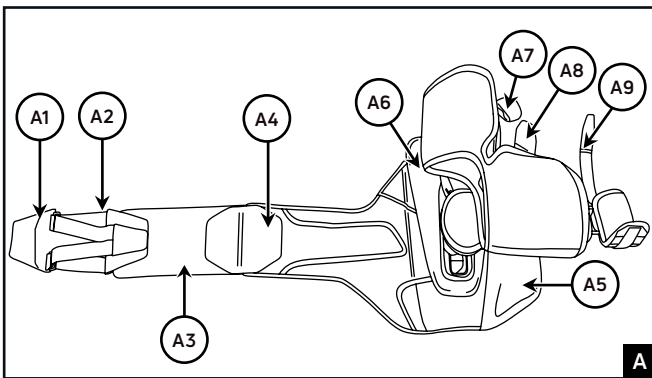

## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

**INTENDED USE/INDICATIONS:** Soft-tissue strains; Anterior shoulder dislocation; Posterior shoulder dislocation; Non or minimally displaced proximal humerus fractures; Low-grade AC-Joint dislocations; Non or minimally displaced clavicle fractures; Brachial plexus injuries or other paralytic or neurologic injuries to the upper extremity; Pectoral tendon rupture and distal biceps rupture; Rotator cuff repairs; Bankart and capsular shift procedures; Bone block or Latarjet reconstructions for anterior shoulder instability; Shoulder replacement CRIF or ORIF of proximal humerus fractures; AC-Joint stabilization of acute and chronic; AC-Joint instabilities; Proximal biceps tenodesis, SLAP-Repairs; ORIF clavicle fractures; reverse shoulder arthroplasty; Humeral shaft fractures; Scapular fractures; SC joint separation, UCL recon of the elbow and other elbow fracture indications.

### PERFORMANCE CHARACTERISTICS:

Providing immobilization or controlled movement of the limb or body segment.

### CONTRAINDICATIONS:

None

### WARNINGS AND PRECAUTIONS:

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or packaging has been opened.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### COMPONENTS:

#### A. WAIST BELT

- A1. Belt Closure
  - A2. Belt Adjustment Strap
  - A3. Bridge Wing
  - A4. Bridge Adjustment Tab
  - A5. Front Panel
- ARMBRACE
- A6. Rigid Frame
  - A7. Bicep Strap
  - A8. Forearm Strap
  - A9. Hand Strap

#### B. ADJUSTMENTS

- B1. AB Buttons
- B2. IR/ER Button
- B3. IR/ER Dial
- B4. Hand Support
- B5. Forearm Section
- B6. Brace Lock

#### OPTIONAL ACCESSORIES (INCLUDED)

- 1. Armsling Strap
- 2. Wrist Pad

**SIZING:** Measure patient's waist circumference 1" (2.5cm) above belly button.

SIZE	CIRCUMFERENTIAL MEASUREMENT
Universal	24" - 45" (61 - 114cm)

\*For waist sizes 46" - 60" (114-152 cm), adhere BRIDGE WING EXTENSION.

### SIZING THE WAIST BELT:

- To make WAIST BELT larger, reposition BRIDGE ADJUSTMENT TAB towards the end of BRIDGE WING. To make WAIST BELT smaller, reposition BRIDGE ADJUSTMENT TAB closer to BELT CLOSURE.
- For petite patients, remove BRIDGE ADJUSTMENT and BRIDGE WING entirely. Reapply BELT ADJUSTMENT STRAP to the WAIST BELT.

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## APPLICATION INFORMATION:

**Preparing the brace:** Prepare brace by opening the BELT CLOSURE TAB.

### Applying brace:

1. Wrap WAIST BELT around waist and attach BELT CLOSURE to FRONT PANEL. Ensure BELT CLOSURE is centered on FRONT PANEL. The BRACE should be situated with lower portion of RIGID FRAME sitting approx. 1 -2" above belly button and ARMBRACE aligned with shoulder. (Figure C)
2. Using BELT ADJUSTMENT STRAP, pull strap across waist to tighten WAIST BELT. (Figure D)
3. To secure ARMBRACE, adhere BICEP CLOSURE TAB. To tighten, pull bicep adjustment strap across bicep. (Figure E)
4. Repeat Step 3 for FOREARM and HAND STRAPS.

### BRACE MODIFICATION INFORMATION: (TO BE PERFORMED BY A LICENSED MEDICAL PROFESSIONAL ONLY)

**\*Change abduction angle:** Push in both AB BUTTONS at the same time. Adjust PANEL by 15° increments until appropriate angle is achieved. Release buttons. Ensure both buttons are retracted outward and clicked into position. (Figure F)

**\*Change internal / external angle:** While depressing IR/ER BUTTON, twist IR/ER DIAL (Figure G) counter-clockwise until loose enough to rotate ARMBRACE to appropriate angle. Retighten DIAL by twisting clockwise to lock brace into position.

**\*Change arm sling brace height:** While depressing the IR/ER BUTTON, twist IR/ER DIAL counter-clockwise until loose enough to move ARMBRACE up or down the PANEL until appropriate position is achieved. (Figure G) Retighten DIAL by twisting clockwise to lock brace into position.

**Hand support:** Extend HAND SUPPORT to achieve proper hand position and support. (Figure H) For a more comfortable fit, bend lower portion of HAND SUPPORT to fit lower palm. **NOTE:** Apply optional WRIST PAD, if needed.

**Elbow extension and flexion - Option 1:** Retract HAND SUPPORT back into ARMBRACE until HAND SUPPORT contacts FOREARM SECTION. Freely flex and extend lower arm. **NOTE:** If optional WRIST PAD is attached, remove.

**Elbow extension and flexion - Option 2:** To extend and flex elbow for singular motion exercises, push BRACE LOCK toward wrist to unlock. Extend and flex elbow. Brace will lock into position at 90°.

**Optional arm sling strap:** Adhere strap end with buckle onto front of WAIST BELT above BELT CLOSURE TAB. Secure remaining strap end across chest and adhere to rear opposite side of WAIST BELT adjacent to AB buttons. (Figure I)

**CLEANING INSTRUCTIONS:** Separate the rigid components from the soft goods prior to cleaning.

Hand wash in water (86°F/30°C) using soap. Air dry.

**MATERIAL CONTENTS:** Nylon, Aluminum, Stainless Steel, Zinc coated steel wire, Copper, Nylon woven, Polyurethane Foam, Silicone, Epoxy, Polyurethane

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

**RX ONLY.**

**FOR SINGLE PATIENT USE ONLY**

**NOT MADE WITH NATURAL RUBBER LATEX.**

**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

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- B1. Przyciski AB