
AirHeel™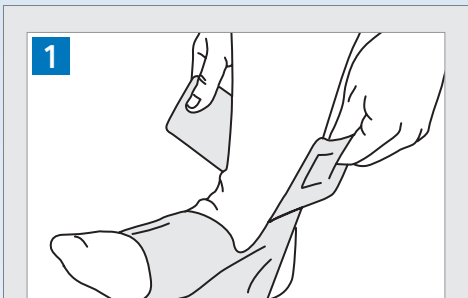

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS:

Helps alleviate pain associated with plantar fasciitis. Providing mild protection of a limb or body segment for acute and prophylactic care.

CONTRAINDICATIONS: N/A

WARNINGS AND PRECAUTIONS:

- Do not use over open wounds.
- Do not use this device if it was damaged and/or the package has been opened.
- For maximum comfort and support, wear an absorbent sock and a laced athletic shoe when wearing the brace.
- If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

1) Insert foot into AirHeel. Position heel in opening in back of wrap. Position Achilles aircell over tendon. The height of the Achilles aircell may be repositioned by detaching and firmly reattaching in place. (Fig. 1)

2) Place foot flat on floor and secure ankle strap. Make sure that foot aircell is located under arch of foot. To adjust AirHeel fit, tighten/loosen ankle strap. (Fig. 2)

CLEANING INSTRUCTIONS:

Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

FOR SINGLE PATIENT USE ONLY.

 NOT MADE WITH NATURAL RUBBER LATEX.

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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