ENGLISH
BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:
The intended user should be a licensed medical professional, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INDICATIONS

The DonJoy PM II Wrap Long is designed to provide support for the relief of pain following injuries such as:
 - Mild to moderate MCL and/or LCL sprains
 - Mild to moderate ACL and/or PCL instabilities
 - Meniscal repairs
 Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

- Unfasten and loosen the straps at the thigh and calf.
- Pull the sleeve onto the leg using the loops at the top of each hinge bar. If using wrap-around version, simply wrap around leg and fasten over the front of the leg. Position the sleeve so that the hinges are parallel to each other, and the top of the circular cut-out aligns with the top of the kneecap. (Fig. 1)
- Fasten the lower calf strap first (#1), then the upper thigh strap (#2). (Fig. 2)
- Secure the lower thigh strap (#3) (Fig. 3) and then the upper calf strap (#4) (Fig. 4) on the back of the leg.

CONTRAINDICTIONS: N/A

HINGE ADJUSTMENT INSTRUCTIONS
Action Hinge: With hinges slightly flexed, flip open hinge cover, change pins to desired settings and close cap. To avoid damage to hinges, both flexion pins must be set in identical the position.

USE AND CARE

- Hand wash in cold water using mild soap, rinse thoroughly. AIR DRY Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.
 - The hinge mechanism requires little maintenance in normal use. However, you may apply a dry lubricant such as a Teflon® spray to the internal mechanism of the hinge.

WARNINGS AND PRECAUTIONS




It is recommended that the first application be made under supervision of a health care professional.
 - If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately. Functional knee braces are not recommended to be worn over clothing.
 - Do not use over open wounds.
 - Do not use this device if it was damaged and/or package has been opened.
 - If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
 NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

FOR SINGLE PATIENT USE ONLY.

NOT MADE WITH NATURAL RUBBER LATEX.

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

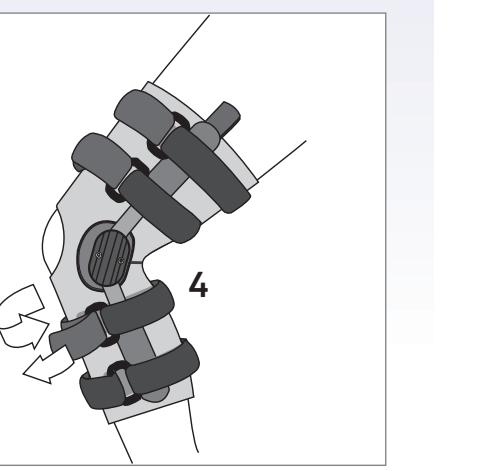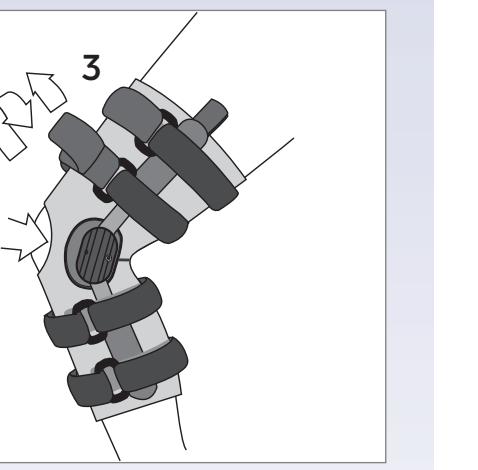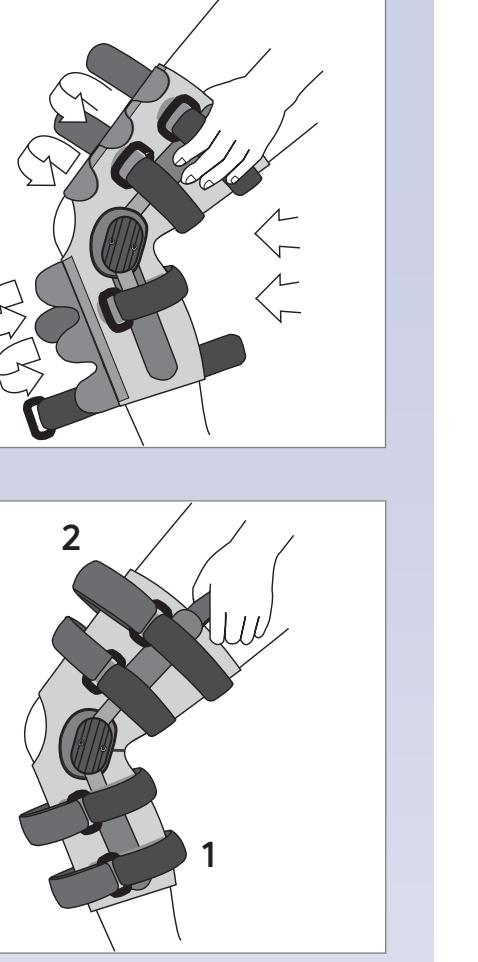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