DONJOY®

**Playmaker™/Playmaker™ II
Knee Brace**

ENGLISH	1	NEDERLANDS.	11	ČEŠTINA	21
DEUTSCH	3	SLOVENČINA	13	PORTUGUESE.	23
FRANÇAIS	5	DANSK	15	NORSK	25
ESPAÑOL	7	SVENSKA	17	简体中文	27
ITALIANO	9	SUOMI	19	日本語	29

CAUTION: Federal law (U.S.A. and Canada) restricts this device to sale, distribution or use by or on the order of a physician.

ENGLISH

PLAYMAKER™/PLAYMAKER™ II KNEE BRACE

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE BRACE.


INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

WARNINGS AND PRECAUTIONS:

• Loss of circulation, patient discomfort and patient re-injury are potential effects caused by device failure. • ROM settings should not be changed without supervision of a medical professional. • This product must be prescribed and fitted by a healthcare professional. • The frequency, duration of use and directions for use should be determined by your healthcare professional. • If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately. • Do not use over open wounds. • The support should be snug but not impair circulation. • Do not use the device if it is damaged and/or the packaging has been opened. **NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.


APPLICATION INFORMATION

- Unfasten and loosen the straps at the thigh and calf.
- Pull the Sleeve onto the leg using the loops at the top of each hinge bar. If using Wraparound version, simply wrap around leg and fasten over the front of the leg. Position the Sleeve so that the hinges are parallel to each other, and the top of the circular cut-out aligns with the top of the kneecap.
- Fasten the lower calf strap first (#1), then the upper thigh strap (#2)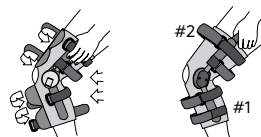


A ACL CONFIGURATION SLEEVE VERSION ONLY

- With the knee bent at 45°, push both condyles (hinges) toward the back of the leg, as shown.
- Maintaining the position of the hinges, secure the lower thigh strap (#3).
- Next, tighten the upper calf strap (#4)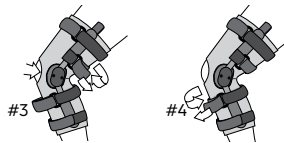


B PCL CONFIGURATION SLEEVE VERSION ONLY

- Maintaining the position of the hinges, secure the upper calf strap (#3).
- Draw the hinges forward slightly before fastening the front thigh strap (#4). Do not pull the hinges beyond midline of the leg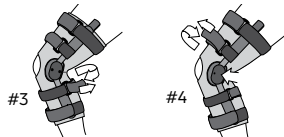

C CI CONFIGURATION SLEEVE AND WRAPAROUND VERSIONS

- With the knee bent at 45°, push both condyles (hinges) toward the back of the leg, as shown.
- Maintaining the position of the hinges, secure the lower thigh strap (#3) and then the upper calf strap (#4) on the back of the leg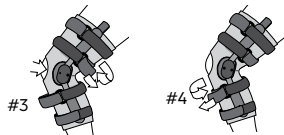

HINGE ADJUSTMENT INSTRUCTIONS

PLAYMAKER™/PLAYMAKER™ II KNEE BRACE with Polycentric Hinge


Indications for use: The DonJoy Playmaker/Playmaker II brace with Polycentric Hinge is designed to support the knee following mild MCL and/or LCL instabilities, mild ACL and/or PCL instabilities, and meniscal repairs. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

Contraindications: Moderate to Severe ligament instabilities. Do not use if you are allergic to any of the materials contained in this product. Your brace comes with 10° extension stops installed.

To change the installed stops, first completely remove the screw as shown. Then remove the stops. Insert the new stops, align with the holes in the hinge plate, then reinsert the screw. Using a Phillips head screwdriver, ensure that the screws are seated in the recess and completely tightened. Both medial and lateral stops must be identical in degree.

Extension Stops: 0°, 10°, 20°, 30°, and 40°.

Flexion Stops: 0°, 45°, 60°, 75°, and 90° (no flexion stops are installed upon shipment)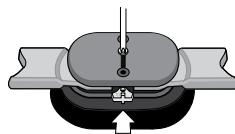


PLAYMAKER II WITH X-ROM HINGE

Indications for use: The DonJoy Playmaker II with X-ROM Hinge is designed to support the knee following meniscal repairs and ACL/ PCL reconstructions. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

Contraindications: Activities requiring full range of motion. Do not use if you are allergic to any of the materials contained in this product.

INSTRUCTIONS FOR HINGE ADJUSTMENT

Adjust the hinge by pulling the Flexion and Extension stops outward and rotating until the desired angle aligns with the button center. Release stops and ensure they are fully inserted. Utilize the quick-lock feature at -10, 0, 10, 20, or 30 degrees, by aligning red arrow with desired degree and sliding the red quick lock button inward until the lock is engaged. Ensure uprights do not rotate. If another lock setting is required, ensure both Flexion and Extension buttons are set at the same angle setting.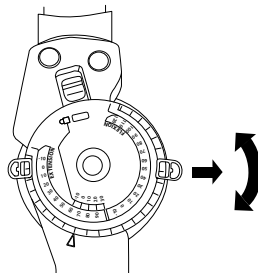

PLAYMAKER™/PLAYMAKER™ II KNEE BRACE with Fourpoint Hinge

Indications for use: The DonJoy Playmaker II brace with Fourpoint Hinge is designed to support and protect the knee following mild MCL, LCL or ACL instabilities and meniscal repairs. Soft-good/ semi-rigid designed to restrict motion through elastic or semi-rigid construction.

Contraindications: PCL instabilities, moderate to severe ACL, MCL, and PCL instabilities. Do not use if you are allergic to any of the materials contained in this product.

INSTRUCTIONS FOR HINGE ADJUSTMENT

Resistance Adjustment Screws (A)

Remove allen wrench from inside of bag. Remove and re-insert adjustment screw into desired location according to patient's comfort and physician's rehabilitation protocol (1-low, 2-medium, 3-high). Push adjustment screw down until threads contact surface of hinge (A1) turn Allen key to tighten adjustment until top of screw is flush with hinge surface (A1). Do not over tighten. Repeat procedure for other hinge. Both hinges must be on the same setting.


Changing Standard External Flexion Stops (Fig. B)

Completely remove screw. Remove the stop and insert the new stop. Align with the hole in the hinge plate, reinsert the screw, and tighten. Repeat procedure for other hinge. Both hinges must have the same stop.

Turn Resistance On And Off. (Fig. C)


Place brace in flexion, insert tool into on/off slot (C1) and slide over. The "0" represents the brace is "off" or has no resistance. When moved away from "0" the brace is "on" or has resistance. Once adjustment is made to both hinges, replace tool in slot.

A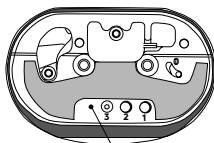


A1-Surface of Hinge

B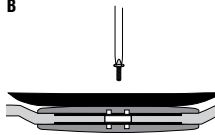

C

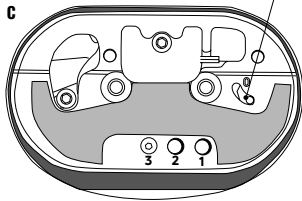C1-On/Off

PLAYMAKER WITH DROPLOCK HINGE

Indications for use: The DonJoy Playmaker with Droplock Hinge is designed to support the knee during ACL rehabilitation. Soft-good/ semi-rigid designed to restrict motion through elastic or semi-rigid construction.

Contraindications: Severe quad deficiency and knee pathology. Do not use if you are allergic to any of the materials contained in this product.

LOCKING AND UNLOCKING DROPLOCK: To engage DropLock simply straighten leg. DropLock will engage 10° before full extension. To disengage DropLock, sit with leg fully extended and press buttons on both hinges. (Fig. D) CAUTION: Before walking in brace, be sure both hinges have fully locked. To disengage hinge, sitting is recommended.


FREING HINGE FROM LOCKING DEVICE:

While DropLock is disengaged, rotate the lever around from the locked position to unlocked position so that the DropLock button is covered by lever. (Fig. E) Before walking, be sure both hinges have button covered by lever. To engage DropLock again, first sit down and straighten leg. Next, take a pen and push the button down while turning the lever to the unlock position. (Fig. F)


D

E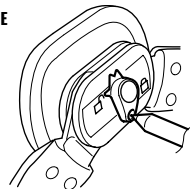

F

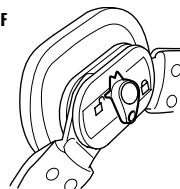USE AND CARE:

- If during activity, re-tightening is necessary, re-tighten all straps following steps in the application instructions.
- All straps and pads may be trimmed to fit individual variations in limb circumference.
- The brace may be used in fresh or salt water. Following use in water activities, drain thoroughly and rinse with clean, fresh water and then air dry.
- Hand wash in cold water (86 °F/30 °C) with mild detergent. AIR DRY ONLY.
- The hinge mechanism requires little maintenance in normal use. However, you may apply a dry lubricant such as a Teflon® spray to the internal mechanism of the hinge. Dry lubrication is recommended following use in water.
- Periodically inspect hinge screws for tightness and retighten as needed.

INTENDED FOR USE ON A SINGLE PATIENT.

NOT MADE WITH NATURAL RUBBER LATEX

USED MATERIAL

Neoprene, Polyester, Nylon, Spandex Lycra, Aluminum, Steel, Polypropylene

WARRANTY: Congratulations on your purchase of a DonJoy® brand brace. Please be assured that it is made of the finest material and technology available. Your brace comes with a 6 month warranty against defects on material and workmanship. Replacement straps, liners, etc., can be purchased from your local representative, medical provider, or by calling Customer Care at (800) 336-6569, fax (800) 936-6569. The warranty, expressed and implied, shall not apply to any brace which has been repaired and/or serviced by anyone other than DJO personnel or by a person authorized to perform such activities by DJO. Any modification of brace will result in termination of warranty.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: Functional knee braces are not recommended to be worn over clothing. Consult your licensed Healthcare professional for your activity limitations.

DJO, Never Stop Getting Better and Playmaker are trademarks of DJO, LLC

FOR SINGLE PATIENT USE ONLY.

RX ONLY.

PLAYMAKER™/PLAYMAKER™ II KNEIORTHESE

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MDSS GmbH
MDSS CH GmbH