ENGLISH
BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS
The DonJoy Conforstrap® is designed to provide comfort and support of the lumbar spine. It may be suitable for symptomatic relief of acute and chronic low back pain. The optional hot/cold pack within the soft-wrap may be used to further aid symptom relief. **Performance Characteristics:** Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS
If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional. Do not use if you are allergic to any of the materials contained within this product. Administration of cold therapy is contraindicated for patients who have previously developed induced hypertension during cold treatment, who have allergies to cold or who have Raynaud's syndrome, peripheral vascular disease or sickle cell anaemia.

WARNINGS AND PRECAUTIONS: For maximum efficacy, ensure to select correct size of strap . If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately . Do not use over open wounds . Do not use this device if it was damaged and/or package has been opened. **NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS
Position the belt around the waistline and close using contact closure. (Fig. 1,2 & 3)
The back support is delivered with a removable mesh textile pocket and a re-usable Hot/Cold pack.
Conforstrap is also delivered with an optional dorsal pad. (Fig. 4). Use either the hot/cold pack or the dorsal pad by attaching it to the inside of the back support.

HOT/COLD PACK - INSTRUCTIONS FOR USE
DIRECTIONS FOR COLD THERAPY USE
1. Place gel compress in your freezer for at least 1 hour.
2. Remove gel compress from your freezer.
3. Place the compress in the removable textile pocket delivered with the back support.
Position the textile pocket containing the compress on the internal part of the back support.
Fix the textile pocket and compress using the hook and loop strap located on exterior of textile pocket.

DIRECTIONS FOR HOT THERAPY USE (BASED ON A 1200 WATT MICROWAVE)
1. Gel compress must be at room temperature before microwave use.
2. Lay gel compress flat in microwave. Heat at full power for 45 seconds.
3. Remove from microwave and check for desired temperature. (Temperature will continue to rise slightly).
4. If additional heat is desired, return to microwave and heat in 10 second increments.
5. Excessive heating may cause gel compress to rupture and leak.
6. Remove gel compress from microwave.
7. Place the compress in the removable textile pocket delivered with the back support.

Position the textile pocket containing the compress on the internal part of the back support.
Fix the textile pocket and compress using the hook and loop strap located on exterior of textile pocket.

HOT/COLD PACK WARNINGS & PRECAUTIONS
To avoid frostbite, cold therapy should not exceed 20 minutes at a time. Watch gel compress while heating. If gel compress starts to expand, turn off microwave. Do not apply gel compress directly to skin to injure skin. Use only as directed. Individuals with circulatory problems should consult a physician before using this product. Do not use this product with any topical pain relievers or ointments. This product is non-toxic, however, it is not intended to be eaten. If additional heat is required, wait 1 hour before reheating gel compress. Excessive heating may cause gel compress to rupture and leak. Keep out of reach of children.

CARE
Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

COMPOSITION
Polyamide, Polyester, PVC, Elasthanne, Acier

LATEX NOT MADE WITH NATURAL RUBBER LATEX.

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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