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DonJoy Combined Instabilities Knee Braces

COMBINED INSTABILITIES KNEE BRACE APPLICATION INSTRUCTIONS

Before applying the brace, please read these instructions completely and carefully. Correct application is vital to the proper functioning of the brace.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INDICATIONS: The DonJoy Combined Instabilities Knee Brace is designed to support the knee and provide protected range of motion following ACL/PCL injuries, ACL/PCL reconstructions and MCL and/or LCL sprains. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction. Providing mild protection of a limb or body segment for acute and prophylactic care.

CONTRAINDICATIONS: None.

APPLICATION INSTRUCTIONS:

A While sitting on the edge of a chair, step through the brace. Flexing the knee to 45°, position the brace as shown in figure 1, centering the hinge about 1" (2.54 cm) above the top of the kneecap and slightly past midline of the leg. Make sure the brace is not twisted on the leg.

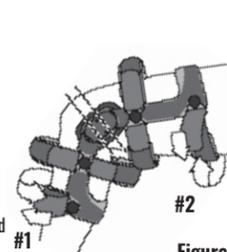
Figure 1

B Secure the bottom calf strap first (#1), then the top thigh strap (#2).

C Keeping the knee bent at 45 degrees, push both condyles (hinges) toward the back of the leg, as shown in figure 2.

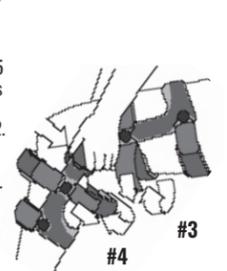
Figure 2

D Maintaining the position of the hinges, secure the lower thigh strap (#3) and then the upper calf strap (#4) on the back of the leg.

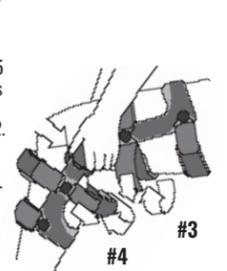
Figure 3

Draw the hinges forward slightly before fastening the front tibial (shin) (#5) and the front thigh strap (#6). Do not pull the hinges beyond midline of the leg.

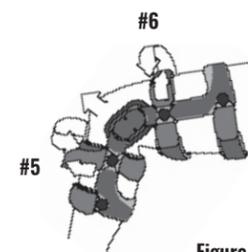
Figure 4

F When all strapping is complete, the hinge should center just above the top of the kneecap and slightly behind midline.

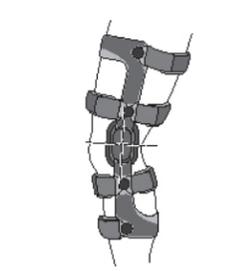
Figure 5

To ensure that no slippage occurs during activity, it is recommended to do a few knee bends or walk for a few minutes, then re-tighten the straps following steps B-E.

INSTRUCTIONS FOR HINGE ADJUSTMENT:

Your brace comes with 10° extension stops installed. To change the installed stops, first completely remove the screw as shown (for Defiance, Legend, DonJoy SE 4-Point figure 5, for 4TITUDE figure 6). Then remove the stops. Insert the new stops, align with the holes in the hinge plate, then reinsert the screw. Using a Phillips head screwdriver, ensure that the screws are seated in the recess and completely tightened. Both medial and lateral stops must be identical in degree.

Figure 6

Lateral slide shield hinge cover (standard on LEGEND and DEFIANCE extra)
In order to remove the extension stop, it is necessary to flex the hinge and slide the protective shield cover off before removing the stop (figure 7).

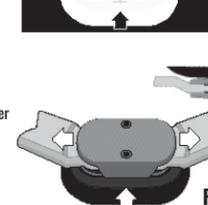
Figure 7

Extension Stops: 0°, 10°, 20°, 30°, and 40°.
Flexion Stops: 0°, 45°, 60°, 75°, and 90° (no flexion stops are installed upon shipment)

USE AND CARE:

- If during activity, re-tightening is necessary, re-tighten all straps following steps in the application instructions.
- All straps and pads may be trimmed to fit individual variations in limb circumference.
- The brace may be used in fresh or salt water. Following use in water activities, drain thoroughly and rinse with clean, fresh water and then air dry.
- Hand wash your liners and condyle pads in cold water with mild detergent. AIR DRY ONLY. DO NOT HEAT DRY LINERS OR PADS.
- The hinge mechanism requires little maintenance in normal use. However, you may apply a dry lubricant such as a Teflon® spray to the internal mechanism of the hinge. Dry lubrication is recommended following use in water.
- Periodically inspect hinge screws for tightness and retighten as needed.

WARNINGS AND PRECAUTIONS:

- Loss of circulation, patient discomfort and patient re-injury are potential effects caused by device failure.
 - ROM settings should not be changed without supervision of a medical professional.
 - This product must be prescribed and fitted by a healthcare professional.
 - The frequency, duration of use and directions for use should be determined by your healthcare professional.
 - If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
 - If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
 - Consult your medical professional immediately if you experience leg pain, tenderness, swelling or redness of the skin as these may be symptoms of a venous thromboembolism.
 - The support should be snug but not impair circulation.
 - Do not use over open wounds.
 - Do not use the device if it is damaged and/or the packaging has been opened.
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

FOR SINGLE PATIENT USE ONLY. **NOT MADE WITH NATURAL RUBBER LATEX. RX ONLY.**

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: Please be advised that dj Orthopedics, LLC, does not recommend simultaneous wearing of functional knee braces bilaterally (right and left legs), during activities (ie . . . skiing) that may cause the braces to contact each other, resulting in potential loss of control and injury. Functional knee braces are not recommended to be worn over clothing. Consult your licensed Healthcare professional for your activity limitations.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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