ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

© 2014 MalleoForce, LLC. All rights reserved. MalleoForce is a registered trademark of MalleoForce, LLC.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

© 2014 MalleoForce, LLC. All rights reserved. MalleoForce is a registered trademark of MalleoForce, LLC.


INTENDED USE/INDICATIONS

- Support and compression following acute and chronic ankle injury and mild ankle and foot osteoarthritis
- Proprioceptive and neuromuscular stimulation through compression (to support balance conditions and movements)
- Mild (grade I) ankle sprains and strains
- Mild ankle instability
- Prevention of injury

Performance characteristics:

- Provides compression (French Standard ASQUAL Class III: 20-36 mmHg and German Standard RAL Class II: 23-32 mmHg) of limb or body segment through elastic construction.
- Provide mild protection of a limb or body segment for acute and prophylactic care

© 2014 MalleoForce, LLC. All rights reserved. MalleoForce is a registered trademark of MalleoForce, LLC.

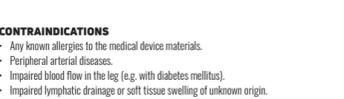© 2014 MalleoForce, LLC. All rights reserved. MalleoForce is a registered trademark of MalleoForce, LLC.

© 2014 MalleoForce, LLC. All rights reserved. MalleoForce is a registered trademark of MalleoForce, LLC.

- CONTRAINDICATIONS**
- Any known allergies to the medical device materials.
 - Peripheral arterial diseases.
 - Impaired blood flow in the leg (e.g. with diabetes mellitus).
 - Impaired lymphatic drainage or soft tissue swelling of unknown origin.
- WARNINGS AND PRECAUTIONS**
- The product is intended for single patient use only.
 - If you have skin conditions/fruama in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
 - Do not apply on open wounds.
 - The product is designed to be worn directly on the skin.
 - Do not apply any lotions or ointments on the relevant part of the body while wearing the support.
 - Remove the support during longer periods of rest.
 - If patients experience pain, swelling, changes in sensation or other unusual reactions occur while using this product, they should contact their doctor immediately.
 - Do not use this device if it is damaged and/or packaging has been opened.
 - Do not use this product in any way that may compromise its integrity.
 - Handle with care: avoid fastening the strap's hook closure onto the knitted sleeve as it may cause damage to the appearance of the product when strap is removed.
 - Keep out of the reach of children as improper use can cause a hazard.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

© 2014 MalleoForce, LLC. All rights reserved. MalleoForce is a registered trademark of MalleoForce, LLC.

© 2014 MalleoForce, LLC. All rights reserved. MalleoForce is a registered trademark of MalleoForce, LLC.

Strap application

To prevent supination (ankle rolling to the outside), check figures A to D. To prevent pronation (ankle rolling to the inside), check figures E to H.

© 2014 MalleoForce, LLC. All rights reserved. MalleoForce is a registered trademark of MalleoForce, LLC.

CLEANING INSTRUCTIONS

Machine washable at gentle cycle (max 40°C) with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Iron off. Hook and loop should be closed for washing.

© 2014 MalleoForce, LLC. All rights reserved. MalleoForce is a registered trademark of MalleoForce, LLC.

© 2014 MalleoForce, LLC. All rights reserved. MalleoForce is a registered trademark of MalleoForce, LLC.

FOR SINGLE PATIENT USE ONLY.

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

© 2014 MalleoForce, LLC. All rights reserved. MalleoForce is a registered trademark of MalleoForce, LLC.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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