ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS:

Elastic knitted ankle support with pneumatic aircells providing intermittent compression on the Achilles tendon to provide systematic pain relief for conditions such as Achilles tendinitis and Plantar Fasciitis. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

FEATURES:

- Interconnected pneumatic aircells
- The aircells provide a massage effect on the Achilles tendon area
- 3-D anatomically contoured knit provides comfortable, uniform compression
- Breathable material

INDICATIONS:

- Chronic or post-traumatic inflammation of the Achilles tendon area
- Achilles tendinitis and achillogynia (pain in the Achilles tendon area)
- Plantar Fasciitis

CONTRAINDICATIONS:

Do not use if you are allergic to any of the materials contained in this product.

WARNINGS AND PRECAUTIONS:

- Do not use over open wounds
- Do not use this device if it was damaged and/or package has been opened
- If pain, swelling, changes in sensation or other unusual reactions occur
- When using this product, you should contact your doctor immediately
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

(HIGH ALTITUDE) At high altitudes the aircells will expand beyond their optimal level. Adjust aircell pressure by tightening or loosening support and/or strap.

APPLICATION INSTRUCTIONS:

We recommend wearing a thin sock for ease of application and for hygiene reasons.

1. Sit down to apply the brace
2. Open the Hook & Loop closure at the top of the support, insert the foot in the support toes first, then pull the support over the forefoot and ankle
3. Ensure that the lower aircell is positioned below the heel
4. Close the Hook & Loop closure at the top of the support, ensuring it doesn't feel too tight or too compressive
5. The upper posterior aircells should be positioned on each side of the Achilles tendon
6. Attach the removable strap to the small velours piece on the back of the support. Wrap the strap around the ankle, to provide additional compression. Avoid tightening the strap too much and loosen it immediately if you feel it is too tight.

While walking, the air will circulate between the upper and lower aircells, providing intermittent compression and a massage effect to the Achilles tendon.

Advice: When using the removable strap, avoid direct contact between the Hook & Loop of the strap and the knitted support, as this will damage the visual aspect of the knit.

USAGE INSTRUCTIONS:

- If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues. No jewelry is being used.
- The support should not be worn for more than 8 hours a day
- With proper use and the observance of all instructions, no pressure points should occur.
- The manufacturer shall bear no responsibility for any changes to the product or incorrect usage

CARE:

- Hand wash in lukewarm water (30 °C) with a mild detergent
- Air dry (do not place on radiators/heaters or other heat sources)
- Do not use bleach or chlorine, do not iron.

COMPOSITION:

- Knitted material: Polyamid, elastin, polyester, natural rubber latex
- Internal Cover: terrycloth
- Aircells: Polyurethane film, Polyether / Polyurethane tube, Polyether foam
- Strap: Polyamid

LATEX FOR SINGLE PATIENT USE ONLY. MADE WITH NATURAL RUBBER LATEX.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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