
MalleoForce / MalleoForce +  
AchilloForce Air  
ANZIEHHILFE  
DONNING AID  
DISPOSITIF D'APPLICATION

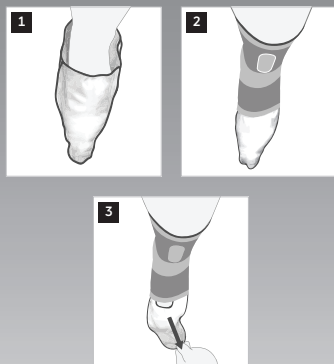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**ENGLISH**

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USER PROFILE:**

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**INTENDED USE/INDICATIONS:**

This device is an optional Donning Aid to be used with the DonJoy MalleoForce / MalleoForce + / AchilloForce Air. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

**CONTRAINDICATIONS:**

Do not use if you are allergic to any of the materials contained in this product.

**WARNINGS AND PRECAUTIONS:**

- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or package has been opened.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to use of this device. (HIGH ALTITUDE) At high altitudes the aircells will expand beyond their optimal level. Adjust aircell pressure by tightening or loosening support and/or strap.

**APPLICATION - DONNING AID:**

1. Place your foot in the Donning Aid.
2. Slide the ankle brace over it.
3. Remove the Donning Aid, by pulling it at the front end of the foot.

**CARE:**

- Hand wash in lukewarm water (30 °C) with a mild detergent.
- Air dry (do not place on radiators/heaters or other heat sources)
- Do not use bleach or chlorine, do not iron.

**COMPOSITION:**

PES, Polyester

**FOR SINGLE PATIENT USE ONLY.**

**NOT MADE WITH NATURAL RUBBER LATEX.**

**WARRANTY** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

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