**1****ENGLISH**

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

**INTENDED USER**

The intended user should be the patient, a licensed healthcare professional, or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

**INTENDED USE/INDICATIONS**

The DonJoy LumboForce™ 2 is designed to provide comfort and support of the lumbar spine. It may be suitable for the symptomatic relief of acute and chronic low back pain.

- Lumbalgia
- Osteochondrosi
- Spondylarthrosi

**2****CONTRAINDICATIONS**

If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a healthcare professional.

**PERFORMANCE CHARACTERISTICS**

- Providing support and stabilization to the limb or body segment.
- Providing partial immobilization or controlled movement to the limb or body segment.
- Providing mild protection of a limb or body segment for acute and prophylactic care.

**WARNING AND PRECAUTIONS**

We recommend that the first application be made under supervision of a healthcare professional. If you have skin conditions/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a healthcare professional. If you experience any discomfort, pain, swelling or sensation changes, or any unusual reactions while using this product, consult your healthcare professional immediately. This product is for single patient multiple use only. If the device is damaged and/or packaging has been opened, the product should be removed during long periods of rest. Do not use during showering or bathing. Keep the product away from children. Do not use on open wounds) and/or sensitive skin. Do not use this product if it is damaged or defective. Do not use this product in any way that may damage its integrity or use the device other than intended. The frequency and duration of use should be determined by a healthcare professional. **NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.**

**3****APPLICATION INSTRUCTIONS**

1. A cotton shirt or vest should be worn underneath the back support. If possible, the back support should be put on while standing.
2. Open the sacrum belt and loosen all straps.
2. The back pad on the outer side of brace can be adjusted vertically.
3. The removable elastomer back pad is fastened by hook-and-loop in the centre of the back support (arrow symbol, pointed upwards), with the stimulating dots facing the back's curvature.
4. Place the lumbar support around the waist. Make sure that the lumbar belt is correctly centred over your spine.
5. Attach the back support by inserting your hands into the comfortable grip pockets, pull to extend the main fabric, and wrap around your waist. The front closure panels should be closely parallel to each other if you are a man and at a slight angle if you are a woman. Then remove your left hand from the grip pocket and, with the right hand, press down firmly on the top front closure panel.
6. Pull the lateral straps forward equally with gentle force until the desired level of support is achieved. Then fasten them to the front closure panel with the hook-and-loop. The length of the straps can be customized by cutting them.
7. The back support should be fastened tightly, fit your body well, and provide comfort and extend the back support should not apply any pressure to the groin when sitting down. A tightening readjustment may be necessary to adapt belt compression and to avoid pressure points on the groin when sitting down.

**4****CLEANING INSTRUCTIONS**

Machine washable at gentle cycle (max 30°C) with a mild detergent. Hook and loop of the straps should be closed for washing to avoid any damage. Do not bleach. Rese throughly. Air dry. Do not machine dry. Do not dry clean. Do not iron.

**5****COMPOSITION**

**Front:** Polyamide, Acrylic Resin, Polyurethane, Polyester, Polyolefin & Polyurethylène.  
**Back:** Polyamide, Elastane, Polyurethane, Polyester, Polypropylene, Steel.  
**Elastomer back pad:** Polypropylene, Polypropylene 30 fiberglass, Synthetic rubber & Styrenic Block Copolymer.

**NOT MADE WITH NATURAL RUBBER LATEX**

WARRANTY D.J.O. LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

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