ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS:

The DonJoy PCL Knee Braces is designed to provide support to the knee following injuries such as:

- PCL Deficiencies
- PCL reconstructions
- Ligament deficiencies

Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

This product was designed to complement the variety of medical treatments common to the above afflictions. The frequency and duration of use should be determined by your prescribing Healthcare professional.

PERFORMANCE CHARACTERISTICS:

Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS: N/A

APPLICATION INFORMATION:

- 1) While sitting on the edge of a chair, step through the brace. Flexing the knee to 45°, position the brace as shown in figure 1, centering the hinge about 1" (2.54 cm) above the top of the kneecap and slightly past midline of the leg. Make sure the brace is not twisted on the leg. Secure the bottom calf strap first (#1), then the top thigh strap (#2).
- 2) Keeping the knee bent at 45 degrees, push both condyles (hinges) toward the back of the leg, as shown in figure 2. Maintaining the position of the hinges, secure the upper calf strap (#3).
- 3) Draw the hinges forward slightly before fastening the front thigh strap (#4). Do not pull the hinges beyond midline of the leg.
- 4) When all strapping is complete, the hinge should center just above the top of the kneecap and slightly behind midline. To ensure that no slippage occurs during activity, it is recommended to do a few knee bends or walk for a few minutes, then re-tighten the straps following steps B-E.

INSTRUCTIONS FOR HINGE ADJUSTMENT:

Your brace comes with 10° extension stops installed. To change the installed stops, first completely remove the screw as shown (for Defiance, Legend, DonJoy SE 4-Point figure 5, for 4TITUDE figure 6). Then remove the stops. Insert the new stops, align with the holes in the hinge plate, then reinsert the screw. Using a Phillips head screwdriver, ensure that the screws are seated in the recess and completely tightened. Both medial and lateral stops must be identical in degree.

Lateral slide shield hinge cover (standard on LEGEND and DEFIANCE extra)
In order to remove the extension stop, it is necessary to flex the hinge and slide the protective shield cover off before removing the stop (figure 7).

Extension Stops: 0°, 10°, 20°, 30°, and 40°.
Flexion Stops: 0°, 45°, 60°, 75°, and 90° (no flexion stops are installed upon shipment)

CLEANING INSTRUCTIONS:

- If during activity, re-tightening is necessary, re-tighten all straps following steps in the application instructions.
- All straps and pads may be trimmed to fit individual variations in limb circumference.
- The brace may be used in fresh or salt water. Following use in water activities, drain thoroughly and rinse with clean, fresh water and then air dry.
- Hand wash your liners and condyle pads in cold water with mild detergent. AIR DRY ONLY. DO NOT HEAT DRY LINERS OR PADS.
- The hinge mechanism requires little maintenance in normal use. However, you may apply a dry lubricant such as a Teflon™ spray to the internal mechanism of the hinge. Dry lubrication is recommended following use in water.
- Periodically inspect hinge screws for tightness and retighten as needed.

Always consult with your physician or therapist before making changes to the brace.

WARNINGS AND PRECAUTIONS:

- This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use.
 - If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
 - Do not use over open wounds.
 - Do not use this device if it was damaged and/or the package has been opened.
 - If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**FOR SINGLE PATIENT USE ONLY.
Rx ONLY**

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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MDSS GmbH